



User Manual



Kit Includes:

Act+Fast neoprene vest - PVC bladder - Airway assembly -Thirty foam plugs - Users Manual - Storage bag *All contents are lead and latex-free*

> 1 year product warranty For instructions e-mail: info@actfastmed.com Act Fast LLC www.actfastmed.com



Directions for Use

Intended for use under the direction of Health or Safety Instructors.

Each kit includes 10 green foam plugs for beginners, and 20 yellow foam plugs for more advanced students.

Foam plugs can be softend by rolling with fingers. Allowing them to be expelled more easily from the trainer, if needed.



providing realistic training, but they should be kept out

of reach from children under 4 years of age.

Pull the vest over your head and adjust the shoulder straps and waist belt to fit snugly.

Drop a foam plug into the airway.



Place your hands in the correct position, between the navel and the rib cage, resting them lightly on the vest.



Then deliver a quick upward thrust to dislodge the obstruction. Do not use force, it is not necessary and may cause injury. Correct technique will result in the plug shooting into the air, providing instant feedback and teaching the concept.



The training vest can be used in a variety of positions to create realistic scenarios, including: standing, sitting and lying on the ground. Also, it allows the student to practice self rescue, using a counter top or chair.

Cleaning Instructions

From the underside of the vest, release velcro straps and remove the PVC bladder and airway assembly from neoprene vest.



Wash vest by hand using mild detergent, hang to air dry.