

Interactive Suture Trainers LF00890U & LF00892U INSTRUCTION MANUAL



ABOUT THE SIMULATOR

This practice and demonstration suture trainer pad should be considered disposable, but with proper care and treatment, it will serve its purpose for a long period of time. The lifelike material can be easily cut and stitched. With a sharp scalpel, wounds can be cut at any desired depth or location on the pad. The pad is easy to remove. Each cut can be repeatedly sutured until skin at any one site becomes so heavily perforated that it will no longer support more suturing.

LIST OF COMPONENTS

- 1jig
- 1 pad
- Nylon suture
- Instrument case
- Scalpel
- Tissue forceps
- 5¹/₂" scissors
- Needle holder
- Blue soft bag

ATTACHING THE PAD ONTO THE JIG

- 1. Take pad and place the right side onto jig.
- 2. Curl right side into jig.
- 3. Look at all four sides of the pad. Make sure all are secure on the jig.
- 4. Place jig onto arm or leg by positioning jig, then fasten tightly with hook-and-loop fasteners.

REMOVING PAD AND JIG

- 1. To remove jig, release hook-and-loop fasteners and slip jig off of arm or leg.
- 2. To remove pad, lift pad from jig and replace with replacement pad when ready. (See Figure 1.)



All kits containing instruments are to be used for trainer purposes only. Instruments are not sterile and not intended for human use.



CARE OF THE SIMULATOR

Do not use providone-iodine prep solutions. These solutions will indelibly stain the pad. Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with soft cloth or paper towels. When not in continuous use, it is recommended to store the simulator in the soft storage case furnished, or cover it with a cloth.

CAUTIONS

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of paper or plastic. These materials will transfer stains that are not removable. Ballpoint pens will also make indelible stains.

AVAILABLE SUPPLIES

LF00891U	Replacement Interactive Suture Pads (Light)
LF00891(A)U	Replacement Interactive Suture Pads (Light) — pkg. of 3
LF00893U	Replacement Interactive Suture Pads (Dark)
LF00893(A)U	Replacement Interactive Suture Pads (Dark) — pkg. of 3
LF09919U	Nasco Cleaner

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

Other Available *Life*/form Simulators

LF01042USuture KitLF01046UFacial Suturing ModulesLF01028USuture Practice ArmLF01034USuture Practice Leg







LF01028U



ONLINE: EMAIL: PHONE: ADDRESS: NascoHealthcare.com lifeform@enasco.com 1.800.431.4310 901 Janesville Ave. Fort Atkinson, WI 53538

111