

## Male Catheter Model LF01150U INSTRUCTION CARD



## **USING THE MODEL**

- Use this model to practice sterile technique and proper cleansing procedures before, during, and after insertion.
- The model will accept any catheter size 16 or smaller.
- Be sure the balloon on the catheter is fully deflated before insertion or removal. Always lubricate the catheter thoroughly before each insertion.
- Lift the penis to straighten the urethra. Gently insert the catheter into the urethral meatus, through the prostate and into the bladder. Proper insertion in a live patient is evidenced by a flow of urine through the tube.



## **CARING FOR THE MODEL**

- The model may be washed in mild soap and water if it becomes soiled.
- Always store it in the plastic bag when not in use.
- Printed material or ballpoint pens will transfer indelible stains to the model. Certain finishes may also be damaged from prolonged contact with the model.

## **AVAILABLE SUPPLIES**

LF01127U	16 FR Foley Catheter
LF01128U	16 FR Foley Catheter, 10-pack
LF00985U	Lubricant Kit, 2 oz. bottles, 6-pack



ONLINE: E-MAIL: PHONE: ADDRESS: NascoHealthcare.com lifeform@enasco.com 1.800.431.4310 901 Janesville Ave. Fort Atkinson, WI 53538