M72 Male Eldery Care Manikin

Feature

The joints are flexible to move.

The body is designed for waterproof.

It is good to practice for shampooing, brushing and drying hair. It is good to practice for changing position, changing of cloth,

cleaning, and moving in a wheelchair. This is appropriate lightweight and carriageable by single person. It is possible to change and to brush the denture. (for M72) It is possible to hold the position naturally as like a eldery person.

The pictures below are showing how they work with learners.

Specification

	M72
Height	168cm
Weight	14kg
Hair	Wig
Material	Soft Resin within Metallic Framework

M72 168cm 14kg DOs

Handle with care

The model is made of special composition of soft resin. Please handle it with the utmost care like an actual baby at all times.

After each training session, dry the model with soft cloth and powder with talcum powder.

Store the model with its clothe on, covered by the supplyed cloth, away from heat, moisture and direct sunlight.

DON'Ts

Never wipe the model by thinner or organic solvent. Don't mark on the model with pen or leave any printed materials in contacted with their surface. Ink marks on the models will not be removable.



KYOTO KAGAKU co., ltd

http://www.kyotokagaku.com e-mail: rw-kyoto@kyotokagaku.co.jp

Main Office and Factory

15 Kitanekoya-cho Fushimi-ku Kyoto 612-8388, Japan Telephone : 81-75-605-2510 Facsimile : 81-75-605-2519

KyotoKagaku America Inc. USA,Canada,and South America 3109 Lomita Boulevard, Torrance, CA 90505,USA Telephone : 1-310-325-8860 Facsimile : 1-310-325-8867