Gluteus Maximus Injection Simulator



This simulator is designed to practice in the three ways of injection into the buttocks. It is made of a special resin that allows this model to be repeatedly injected without leaving a trace. It comes equipped with an indicator that shows the accuracy of syringe positioning and depth.



Features:

- 1. The model is made from three individual layers, which are skin, muscular and skeleton. Therefore it allows identifying the injection spot as well as practicing with a patient.
- 2. The model is designed for three methods: Quarter-Trichotomy, Hohashtayter Method and Clark Method.
- **3.** Attached indicator shows the accuracy of injection. If the injection spot is correct, the syringe can inject liquid.
- It allows palpating the landmarks such as Greater trochanter, Superior anterior iliac spine, Posterior superior iliac spine and Sacrum.
- 5. The skin and the muscular are replacement parts.



Set includes:

- 1 Main Body
- 1 Stand for Hohashtayter Method

Specifications are subject to change.

- 1 Controller
- 1 Power AC Adapter
- 1 Set of Sensor Pads

Replacement Parts:11213-010Skin11213-020Sensor for Quarter-Trichotomy11213-030Sensor for Hohashtayter Method

- 11213-040 Sensor for Clark Method 11213-050 Controller
- 11213-030

11213-020



11213-040