Caution:

Don't mark on the phantom with pen or leave printed materials contacted on its surface. Ink marks on the phantom will be irremovable.

US-9

Ultrasound- Guided Breast Biopsy phantom



Contents

Please read before training	
General information	P. 2
Preparation	
1 Before training	P. 2
2 Training session	P. 2



This product is intended to be used for medical training. Do not use for other than the intended purpose.

Preparation/ Training

General information Training session

Set includes:

Duo Set: 1387-000 (transparent + opaque type) Transparent Set: 11387-100 (1 pair of transparent type) Opaque Set: 11387-200 (1 pair opaque type)



11387-000 (transparent + opaque type)

▲ Caution

- •This is a disposable product.
- The aqueous material used in this product contracts as time passes after the package is opened.
- •After opening the package, store the product along with the case within a sealed bag or a similar container.

(This method is only for temporary storage. You can delay, but cannot stop contraction and deformation of the product by this storage method.)

- •No fluid injection is allowed.
- •Do not stretch or press hard on the puncture area.
- Dispose the product as general waste after use.

1. Before training



①Remove the back cap on the case.





②Remove the film from the case.



③Take out the product from the case while inserting a finger between the case and the product to let the air get into between them, then put the product on the back cap.

2. Training session



①Conduct puncture training under ultrasonic guide after applying water or ultrasonic gel on the product.

*****Temporary storage



After you have removed the transparent film from the case, store the product along with the case after closing the back cap in a sealed bag or similar container. (The product can be stored only for a short while by this method.)



Don't mark **t**he phantom with **a** pen or leave printed materials **in** contact **with** its surface. Ink marks on the phantom will be irremovable.



KYOTO KAGAKU co., ltd

URL : http://www.kyotokagaku.com e-mail : rw-kyoto@kyotokagaku.co.jp

Main Office and Factory 15 Kitanekoya-cho Fushimi-ku Kyoto 612-8388, Japan

Telephone : 81-75-605-2510 Facsimile : 81-75-605-2519

KyotoKagaku America Inc. USA,Canada,and South America 3109 Lomita Boulevard, Torrance, CA 90505,USA Telephone : 1-310-325-8860 Facsimile : 1-310-325-8867